

We are pleased that you want to improve your brain performance with the training HeadApp. With this step-by-step guide, we want to help you get started.

Your therapist has already registered you in HeadApp. Here you will find your login details.

Login-Email:

Password:

[to be entered by therapist for patient]

Start HeadApp on the Internet.

If you have a **Windows or Mac PC or Laptop**, use this link (Ctrl + Click):



- Click on "**Home Version**". You don't have to pay anything because your account has already been set up by your therapist.
- **Enter your Login details** and click on **Login**.
- click on "**Data protection Info**", click on "**I agree and continue**", click on "**Continue**"
- Start the training now.

If you have an **iPad**, use this link and download the App (Apple App Store):



- Open In "App Store"? and click on "**open**"
- Click on "**GET**" (If needed confirm with your PIN or fingerprint) You don't have to pay anything because your account has already been set up by your therapist.
- Click on "**OPEN**"
- Click on "**Home Version**".
- **Enter your Login details** (*e-mail address and Password*) and click on **Login**. Do not "Sign in with Apple"
- click on "**Data protection Info**", click on "**I agree and continue**", click on "**Continue**"
- Start the training now

If you have an **Android Tablet**, use this link and download the App (Google Play Store):



- Open In "**Google Play Store**" and click on "**Install**"
- click directly on "**Open**" or **start by clicking on the HeadApp icon on your desktop**. You don't have to pay anything because your account has already been set up by your therapist."
- Click on "**Home Version**".
- „Allow HeadApp to access photos, media, and files on your device?“ **Click „Allow“**
- **Enter your Login details** (*e-mail address and Password*) received from therapist and click on **Login**.
- click on "**Data protection Info**", click on "**I agree and continue**", click on "**Continue**"
- Start the training now